
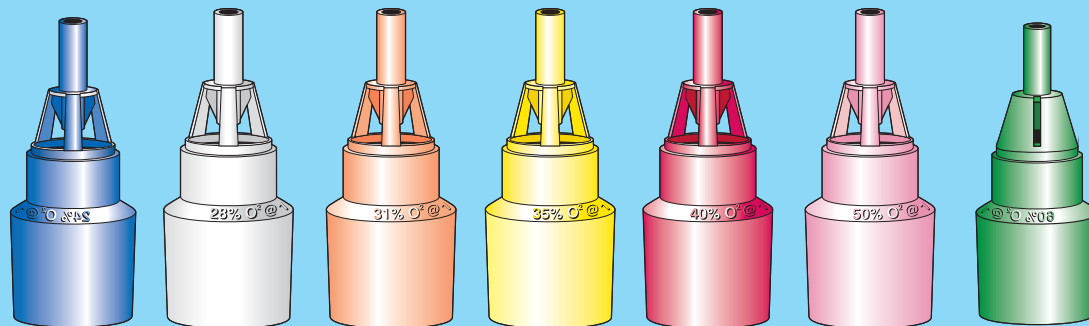


Venturi Diluter Jet Flowrate Chart

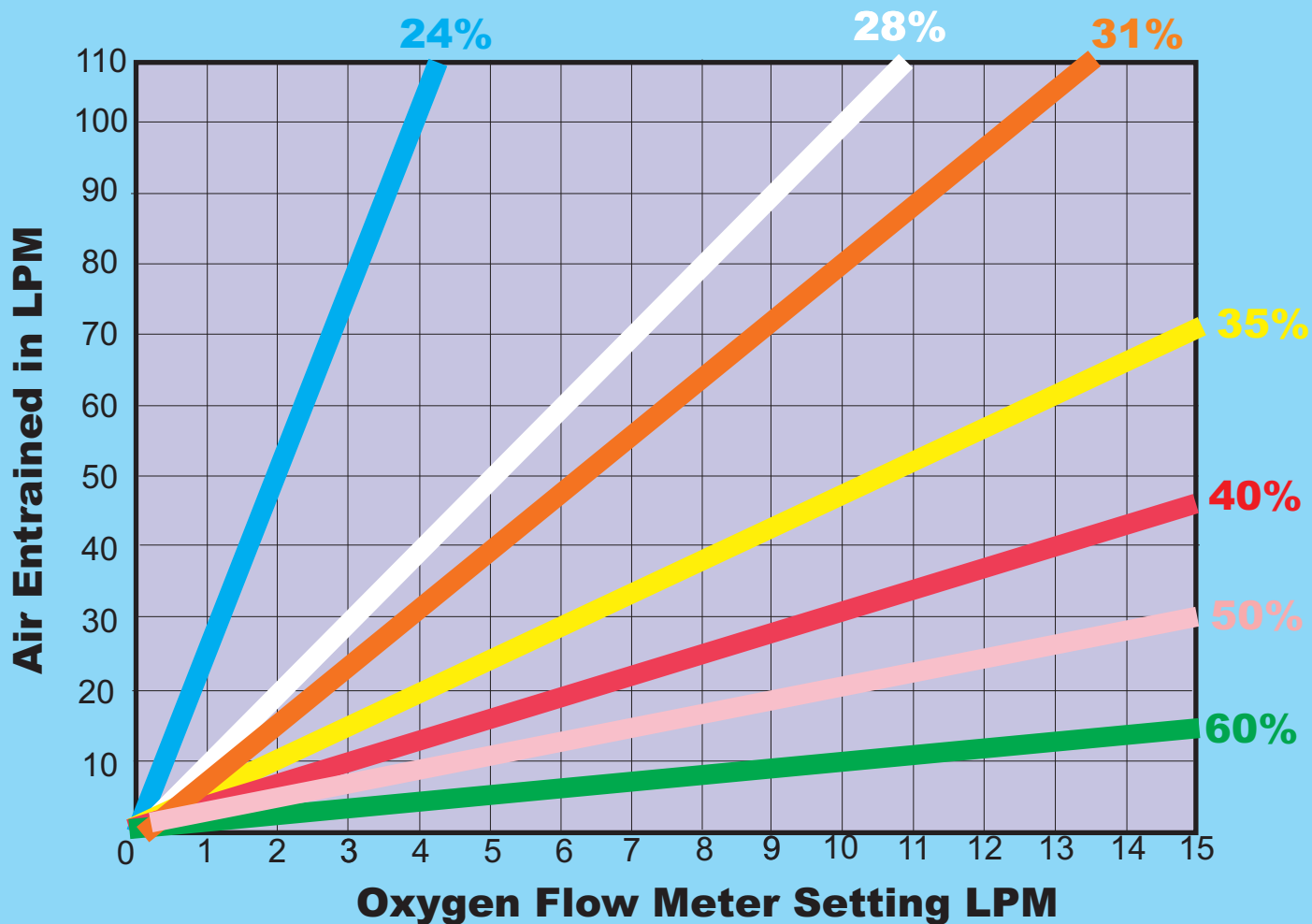

| Oxygen % | 24% | 28% | 31% | 35% | 40% | 50% | 60% |
|-----------------------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|
| Flowmeter Set @ LPM | 2 | 4 | 6 | 8 | 10 | 13 | 15 |
| (** Range) | (1 - 4) | (2 - 6) | (4 - 8) | (6 - 10) | (8 - 12) | (11 - 15) | (14-15) |
| Air Entrained via Jet | 50 | 40 | 46 | 48 | 30 | 28 | 15 |
| Total Flow Delivered | 52 | 44 | 52 | 56 | 40 | 41 | 30 |

**Venturi jets can be set to a range of flows and still deliver accurate oxygen %. Range shown above Total Flow must always be adequate for patient requirement. (Patient Inspiratory Flowrate 25 LPM average)