

Gas sampling Lines

Medical device Class 2A

Intended use

For the sampling of exhaled carbon dioxide (CO₂) from a patient as part of a respiratory or oxygen delivery system.

Intended user

Healthcare professionals trained in respiratory care and monitoring.

Material

Styrene Connectors Luer, Tubing PE/PVC (DEHP FREE)

Materials do not contain phthalates


Materials do not contain latex or substances derived from animals

Instructions for use

- 1) Luer connections on Gas line to be used to connect one end to the patient device at the Luer Port. The other end to be connected to Luer connection on CO₂ Monitor.
- 2) Once connected to CO₂ monitor and patient device check monitor for reading being displayed to confirm correct operation.
- 3) ensure all connections are secure using luer loc screw action. Ensure gas tight seal is achieved.

NOTE

These directions are general guidelines intended for use by qualified medical personnel.

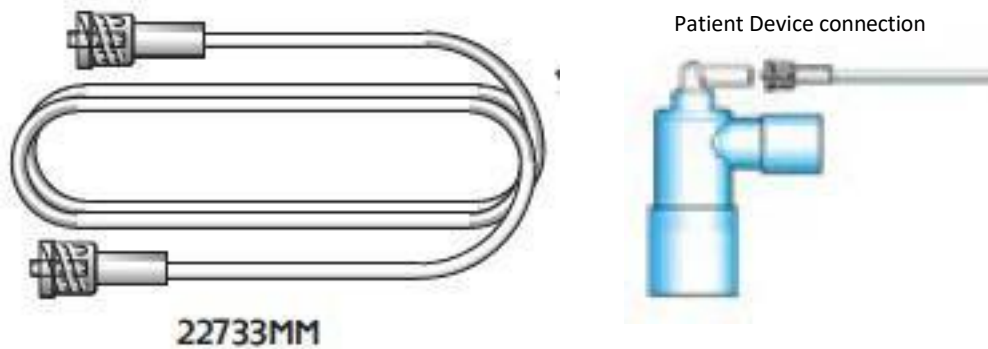

WARNING – FIRE RISK IF OXYGEN IS USED IN THE PRESENCE OF NAKED FLAME OR HEAT SOURCE LIKELY TO CAUSE IGNITION.

Storage

- ⇒ Do not store at extreme temperature or humidity.
- ⇒ Store in clean and dry place.
- ⇒ Do not use if the package open or damaged
- ⇒ Do not store in direct sunlight

Cleaning

DISPOSABLE - FOR SINGLE PATIENT USE ONLY. NOT INTENDED FOR REPROCESSING

Warnings and cautions

1. Be sure all connections are secure.
2. Single-use only. Cleaning or reprocessing for reuse will raise the risk of cross contamination.
3. Meditech Systems Ltd will not take responsibility for use outside of the intended use or by other than the intended users.

Disposal

The devices may be disposed of using local procedures for contaminated medical waste.

The materials are capable of being recycled if a safe process for recycling medical waste is available.

Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

orders@meditechsystems.co.uk

Tel: +44(0)1747 821546

Fax: +44(0)1747 825038

Website; meditechsystems.co.uk

EU Authorised Representative









Clowrey Consultancy







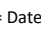

Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire.

Tel: +44(0)7888744362

E-mail: clowreyconsult@gmail.com

Explanation of symbols:

-  = Single Use. The definition of single use is for use on a single patient for a single procedure.
-  = Do not reprocess / re-sterilise
-  = Use by date
-  = Lot or batch number.
-  = Date of manufacture.
-  = Device catalogue number / product code / stock
-  = Caution
-  = Device catalogue number / product code / stock

-  = Manufacturer
-  = Temperature range
-  = Consult instructions for use
-  = Keep away from sunlight
-  = Keep dry
-  = Contains latex
-  = Do not use if packaging is wet or damaged.
-  = Contains or Presence of Phthalates.

This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.
 
 1639 =CE mark and notified body number

ADVERSE INCIDENT REPORTING

Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.