


Anaesthesia Reservoir Bags – Non Sterile

Medical device class; 2A

Intended use

Devices used to provide a reservoir of gas for anaesthesia and ventilator machines. They connector to the machine using a 15mm or 22mm connection to BS EN ISO 5356-1.

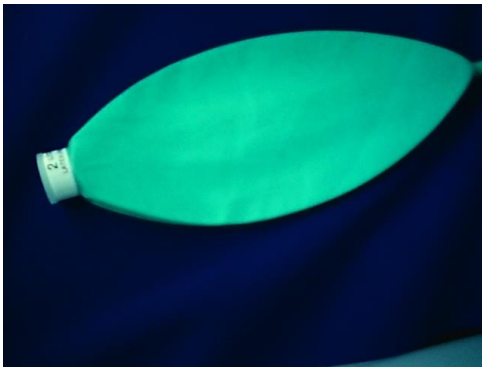

Material

Polychloroprene – latex free

Instructions for use

- 1) Do not use if packaging is damaged or wet
- 2) To be used by personnel trained in anaesthesia /ventilation
- 3) Ensure device is clear of any obstructions before use.
- 4) Ensure device is correct size for the patient
- 5) Ensure the connector in the neck of the bag is secure inside the device

Storage conditions

Storage Temperature & humidity: 10°C (50°F) to 30°C (86°F), 30 – 50% RH

Operating Temperature: -18°C (0°F) to 50°C (122°F)

Sterilization & reprocessing

The devices are supplied non-sterile and are for single use or 7 days where appropriate filtration is added to the breathing system being used. Because of the risk of possible cross contamination these devices must not be reprocessed.

Disposal

Discard the device after use in accordance with local regulation for medical waste.
The device may be incinerated in a facility capable of burning plastic waste.

GMDN #; 34877

Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

orders@meditechsystems.co.uk Tel: +44(0)1747 821546 Fax: +44(0)1747 825038

Facebook: [@meditechsystemslimited](#)
[@MeditechLtd](#)

LinkedIn: [meditech-systems-limited](#)

Twitter:

EU Authorised Representative
















Clowrey Consultancy


Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire.

Tel: +44(0)7888744362

E-mail: clowreyconsult@gmail.com

Explanation of symbols:

- | | | | |
|---|---|---|--|
|  | = Single Use. The definition of single use is for use on a single patient for a single procedure. |  | = Manufacturer |
|  | = Do not reprocess / re-sterilise |  | = Temperature range |
|  | = Use by date |  | = Consult instructions for use |
|  | = Lot or batch number. |  | = Keep away from sunlight |
|  | = Date of manufacture. |  | = Keep dry |
|  | = Caution |  | = Contains latex |
|  | = Device catalogue number / product code / stock |  | = Do not use if packaging is wet or damaged. |
| | |  | = Contains or Presence of Phthalates. |

 1639 =CE mark with notified body number

This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.

ADVERSE INCIDENT REPORTING

Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.