

Safety Bag (resuscitation T) for use with laryngeal mask airways

Intended use

For connection to a laryngeal mask airway (LMA) to provide an oxygen supply indication when the patient is breathing.

Intended user

Healthcare professionals with suitable training in respiratory care.

GMDN; 46867

Material

LLDPE

Materials do not contain phthalates


Materials do not contain latex or substances derived from animals

Instructions for use

Note

These directions are general guidelines intended for use by qualified medical personnel.

1. Connect oxygen tubing to the oxygen connector and set the flow rate prescribed by physician
2. Connect CO2 line to the CO2 Luer connection if required. Then to the co2 monitor
3. Connect the 15mm I.D. connection to the Laryngeal Mask.
4. Oxygen flow will make the indicator bag inflate when connected.
5. Indicator Bag will collapse and re-fill in line with spontaneous breathing of the patient.

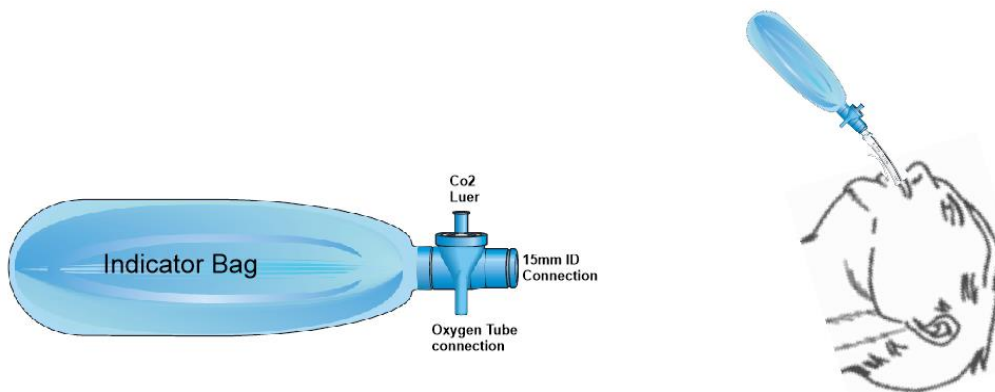

WARNING – FIRE RISK IF OXYGEN IS USED IN THE PRESENCE OF NAKED FLAME OR HEAT SOURCE LIKELY TO CAUSE IGNITION.

Storage

- ⇒ Do not store at extreme temperature or humidity.
- ⇒ Store in clean and dry place.
- ⇒ Do not use if the package open or damaged
- ⇒ Do not store in direct sunlight

Cleaning

DISPOSABLE - FOR SINGLE PATIENT USE ONLY. NOT INTENDED FOR REPROCESSING

Warnings and cautions

1. Be sure all connections are secure.
2. Cleaning or reprocessing for reuse will raise the risk of cross contamination.
3. Meditech Systems Ltd will not take responsibility for use outside of the intended use or by other than the intended users.

Disposal

The devices may be disposed of using local procedures for contaminated medical waste.

The materials are capable of being recycled if a safe process for recycling medical waste is available.

Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

orders@meditechsystems.co.uk

Tel: +44(0)1747 821546

Fax: +44(0)1747 825038

Website; meditechsystems.co.uk

EU Authorised Representative






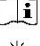











Clowrey Consultancy

Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire.

Tel: +44(0)7888744362

E-mail: clowreyconsult@gmail.com

Explanation of symbols:

	= Single Use. The definition of single use is for use on a single patient for a single procedure.		= Manufacturer
	= Do not reprocess / re-sterilise		= Temperature range
	= Use by date		= Consult instructions for use
	= Lot or batch number.		= Keep away from sunlight
	= Date of manufacture.		= Keep dry
	= Device catalogue number / product code / stock		= Contains latex
	= Caution		= Do not use of packaging is wet or damaged.
	= Device catalogue number / product code / stock		= Contains or Presence of Phthalates.
	= CE mark with notified body number		

This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.

ADVERSE INCIDENT REPORTING

Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.