
MIFU-007-01V

Venturi Masks & Kits

Venturi jets may be fitted to the connector on the aerosol tube or directly to the mask connection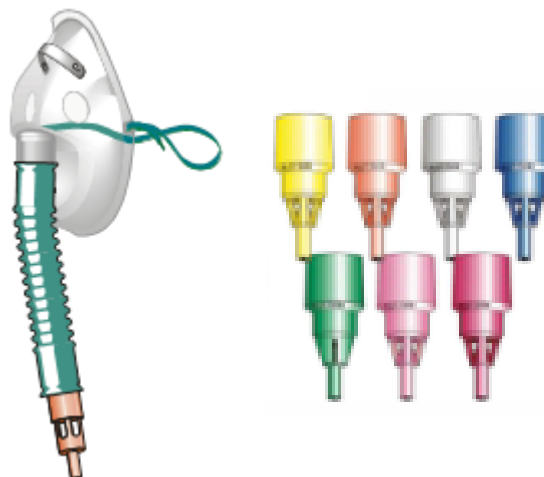

% O ₂	Jet colour Dysans färg Suuttimen väri Dysefarge Dysefarve	O ₂ Flow (lt/min) O ₂ -flöde (l/min) O ₂ -virtaus (l/min) O ₂ -strøm (l/min) O ₂ -flow (l/min)
24	Blue	2
28	White	4
31	Orange	6
35	Yellow	9
40	Red	10
50	Pink	13
60	Green	15

Medical device class; IIA

Intended use

Face mask oxygen delivery with controlled delivery of percentage medical oxygen/air mixture for administration to the patient.

Material

PVC (mask), PP (venturi jet and aerosol tube), Neoprene (strap), Aluminium (Mask nose clip)

Materials do not contain latex or substances derived from animals.

Where phthalates are in the materials the "phthalates" symbol is on the product label.

Instructions for use

Note

These directions are general guidelines intended for use by qualified medical personnel.

1. Choose the Venturi Diluter Jet for the prescribed O₂ percentage required. Attach Jet to mask & attach oxygen supply tubing. Attach oxygen supply tubing to gas source. Set prescribed flow as shown on Venturi diluter Jet.
2. Check for gas flow through the device.
3. Place the mask on the patient's face with the elastic strap below the ears and around the neck.
4. Gently pull the ends of the strap until the mask is secure.
5. Mould the metal strip on the mask to fit across the bridge of the nose.

Storage

- ⇒ Do not store at extreme temperature or humidity.
- ⇒ Store in clean and dry place.
- ⇒ Do not use if the package open or damaged
- ⇒ Do not store in direct sunlight

Cleaning

DISPOSABLE FOR SINGLE PATIENT USE NOT INTENDED FOR REPROCESSING

There is a risk of cross contamination if these devices are reused. They have not been validated for reprocessing or reuse.

Disposal

The devices may be disposed of using local procedures for contaminated medical waste.

The materials are capable of being recycled if a safe process for recycling medical waste is available.

Warnings and cautions

1. Be sure all connections are secure.
2. Product may contain phthalates and could pose a risk for vulnerable patients including pregnant women or infants. Check the device label for the indication of the presence of phthalates.
3. FIRE RISK IF OXYGEN IS USED IN THE PRESENCE OF NAKED FLAME OR HEAT SOURCE LIKELY TO CAUSE IGNITION.

Contraindications when using air entrainment (venturi) jets

1. The assessment of the correct % O₂ must be monitored by checking the patient's blood oxygen levels using equipment such as pulse oximeters or other accurate monitoring devices.
2. Cleaning or reprocessing for reuse will raise the risk of cross contamination.
3. Meditech Systems Ltd will not take responsibility for use outside of the intended use or by other than the intended users

Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

orders@meditechsystems.co.uk

Tel: +44(0)1747 821546

Fax: +44(0)1747 825038

Facebook: [@meditechsystemslimited](https://www.facebook.com/meditechsystemslimited)

LinkedIn: [meditech-systems-limited](https://www.linkedin.com/company/meditech-systems-limited)

Twitter: [@MeditechLtd](https://twitter.com/MeditechLtd)

EU Authorised Representative

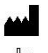
Clowrey Consultancy

Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire.

Tel: +44(0)7888744362

E-mail: clowreyconsult@gmail.com

Explanation of symbols:

	= Single Use. The definition of single use is for use on a single patient for a single procedure.		= Manufacturer
	= Do not reprocess / re-sterilise		= Temperature range
	= Use by date		= Consult instructions for use
	= Lot or batch number.		= Keep away from sunlight
	= Date of manufacture.		= Keep dry
	= Device catalogue number / product code / stock		= Contains latex
	= Caution		= Do not use if packaging is wet or damaged.
	= Device catalogue number / product code / stock		= Contains or Presence of Phthalates.
	1639 =CE symbol and notifiedbody number		

This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.

ADVERSE INCIDENT REPORTING

Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.

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3. Meditech Systems Ltd tager ikke ansvar ved forkert brug eller bruger.