## MIFU-007GS

### Gas sampling Lines

#### Medical device Class IIA

#### Intended use

For connection to patient circuits, masks & other patient end devices where a luer port allows for sampling of the patients CO<sub>2</sub> levels when breathing. Luer connection on gas line connects to CO<sub>2</sub> Port on device and also to CO<sub>2</sub> monitor which measures the CO<sub>2</sub> levels.

#### Intended user

Healthcare professionals with suitable training in respiratory care.

#### Material

Styrene Connectors Luer, Tubing PE/PVC (DEHP FREE)

Materials do not contain phthalates



Materials do not contain latex or substances derived from animals

#### Instructions for use

- 1) Luer connections on Gas line to be used to connect one end to the patient device at the Luer Port. The other end to be connected to Luer connection on CO<sub>2</sub> Monitor.
- 2) Once connected to CO<sub>2</sub> monitor and patient device check monitor for reading being displayed to confirm correct operation.
- 3) ensure all connections are secure using luer loc screw action. Ensure gas tight seal is achieved.

#### Note

These directions are general guidelines intended for use by qualified medical personnel.

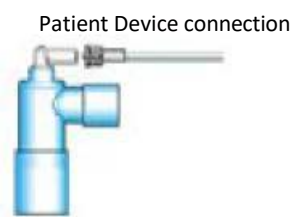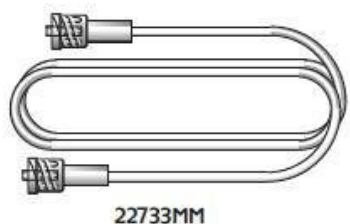**WARNING – FIRE RISK IF OXYGEN IS USED IN THE PRESENCE OF NAKED FLAME OR HEAT SOURCE LIKELY TO CAUSE IGNITION.**

#### Storage

- ⇒ Do not store at extreme temperature or humidity.
- ⇒ Store in clean and dry place.
- ⇒ Do not use if the package open or damaged
- ⇒ Do not store in direct sunlight

#### Cleaning

DISPOSABLE - FOR SINGLE PATIENT USE ONLY. NOT INTENDED FOR REPROCESSING

#### Warnings and cautions

1. Be sure all connections are secure.
2. Single-use only. Cleaning or reprocessing for reuse will raise the risk of cross contamination.
3. Meditech Systems Ltd will not take responsibility for use outside of the intended use or by other than the intended users.

#### Disposal

The devices may be disposed of using local procedures for contaminated medical waste.

The materials are capable of being recycled if a safe process for recycling medical waste is available.

#### Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

[orders@meditechsystems.co.uk](mailto:orders@meditechsystems.co.uk)

Tel: +44(0)1747 821546

Fax: +44(0)1747 825038

Website; [meditechsystems.co.uk](http://meditechsystems.co.uk)

#### EU Authorised Representative

Clowrey Consultancy

Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire. Tel:

+44(0)7888744362

E-mail: [clowreyconsult@gmail.com](mailto:clowreyconsult@gmail.com)

### Explanation of symbols:

	= Single Use. The definition of single use is for use on a single patient for a single procedure.		= Manufacturer
	= Do not reprocess / re-sterilise		= Temperature range
	= Use by date		= Consult instructions for use
	= Lot or batch number.		= Keep away from sunlight
	= Date of manufacture.		= Keep dry
	= Device catalogue number / product code / stock		= Contains latex
	= Caution		= Do not use if packaging is wet or damaged.
	=CE mark with notified body number		= Contains or Presence of Phthalates.

This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.

### ADVERSE INCIDENT REPORTING

Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.

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