## MIFU-014-03

### Safety Bag (resuscitation T) for use with laryngeal mask airways

#### Intended use

For connection to a laryngeal mask airway (LMA) to provide an oxygen supply indication when the patient is breathing.

#### Intended user

Healthcare professionals with suitable training in respiratory care.

#### GMDN; 46867

#### Material

LLDPE

Materials do not contain phthalates


Materials do not contain latex or substances derived from animals

#### Instructions for use

##### Note

These directions are general guidelines intended for use by qualified medical personnel.

1. Connect oxygen tubing to the oxygen connector and set the flow rate prescribed by physician
2. Connect CO2 line to the CO2 Luer connection if required. Then to the co2 monitor
3. Connect the 15mm I.D. connection to the Laryngeal Mask.
4. Oxygen flow will make the indicator bag inflate when connected.
5. Indicator Bag will collapse and re-fill in line with spontaneous breathing of the patient.

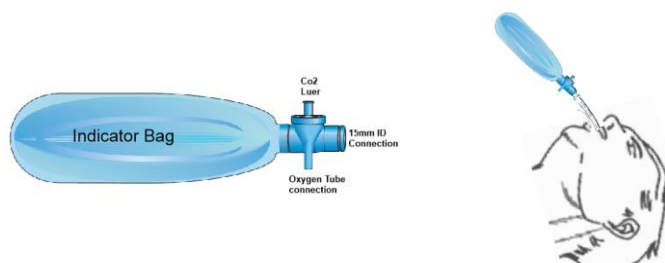**WARNING – FIRE RISK IF OXYGEN IS USED IN THE PRESENCE OF NAKED FLAME OR HEAT SOURCE LIKELY TO CAUSE IGNITION.**

#### Storage

- ⇒ Do not store at extreme temperature or humidity.
- ⇒ Store in clean and dry place.
- ⇒ Do not use if the package open or damaged
- ⇒ Do not store in direct sunlight

#### Cleaning

DISPOSABLE - FOR SINGLE PATIENT USE ONLY. NOT INTENDED FOR REPROCESSING

#### Warnings and cautions

1. Be sure all connections are secure.
2. Cleaning or reprocessing for reuse will raise the risk of cross contamination.
3. Meditech Systems Ltd will not take responsibility for use outside of the intended use or by other than the intended users.

#### Disposal

The devices may be disposed of using local procedures for contaminated medical waste.

The materials are capable of being recycled if a safe process for recycling medical waste is available.

#### Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

[orders@meditechsystems.co.uk](mailto:orders@meditechsystems.co.uk)

Tel: +44(0)1747 821546

Fax: +44(0)1747 825038

Website; [meditechsystems.co.uk](http://meditechsystems.co.uk)

#### EU Authorised Representative

Clowrey Consultancy

Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire.

Tel: +44(0)7888744362

E-mail: [clowreyconsult@gmail.com](mailto:clowreyconsult@gmail.com)

#### Explanation of symbols:

= Single Use. The definition of single use is for use on a single patient for a single procedure.

= Manufacturer

= Temperature range

- |   |  |   |  |
|---|--|---|--|
|  | = Do not reprocess / re-sterilise                |  | = Consult instructions for use               |
|  | = Use by date                                    |  | = Keep away from sunlight                    |
|  | = Lot or batch number.                           |  | = Keep dry                                   |
|  | = Date of manufacture.                           |  | = Contains latex                             |
|  | = Device catalogue number / product code / stock |  | = Do not use of packaging is wet or damaged. |
|  | = Caution  |  | = Contains or Presence of Phthalates.        |
|  | = Device catalogue number / product code / stock |   |  |

CE  
 1639 = CE mark with notified body number

*This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.*

#### ADVERSE INCIDENT REPORTING

Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.

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