# Laryngoscope Blades- sterile

## Medical device Class 1

### Intended use

Devices used to assist in the insertion of artificial airway.

### Material

PE/ PP, Stainless Steel

### Instructions for use

- To be used by personnel trained in CPR & intubation techniques only following accepted CPR or institution approved guidelines for ventilation.
- Ensure device is correct size for the patient's age and weight
- Connect blade to handle and check light bulb or fibre light channel are working correctly before intubation

### Cleaning

#### **DISPOSABLE FOR SINGLE PATIENT USE NOT INTENDED FOR REPROCESSING**

There is a risk of cross contamination if these devices are reused. They have not been validated for reprocessing or reuse.

### Sterilisation

Sterilised by EtO

### Storage and shelf life

Storage Temperature: 10°C (50°F) to 35°C (95°F) and RH 20% - 80%

- Do not store at extreme temperature or humidity.
- Store in clean and dry place.
- Do not use if the package open or damaged
- Do not store in direct sunlight
- The shelf life for sterile devices is marked on the device label. The devices should not be used after this period as the sterile barrier may not maintain sterility after this time.
- For devices in sterile packaging please refer to the storage parameters on the device label.

### Warnings and cautions

- Ensure device is correct size for the patient's age and weight
- Connect blade to handle and check light bulb or fibre light channel are working correctly before intubation

### Disposal

- These devices should be disposed of following the local rules for disposal of contaminated clinical waste.

### Ordering information:

Meditech Systems Ltd, Unit 3 Shrublands Estate, Shearstock, Shaftesbury, Dorset, SP7 9PT, England

[orders@meditechsystems.co.uk](mailto:orders@meditechsystems.co.uk)

Tel: +44(0)1747 821546

Fax: +44(0)1747 825038

Facebook: [@meditechsystemslimited](https://www.facebook.com/meditechsystemslimited)

LinkedIn: [meditech-systems-limited](https://www.linkedin.com/company/meditech-systems-limited)

Twitter: [@MeditechLtd](https://twitter.com/MeditechLtd)

### EU Authorised Representative

Clowrey Consultancy

Address; White House at Bridge, Holycross Village, Thurles, County Tipperary, E41 EC65, Eire.

Tel: +44(0)7888744362

E-mail: [clowreyconsult@gmail.com](mailto:clowreyconsult@gmail.com)

**Explanation of symbols:**

	= Single Use. The definition of single use is for use on a single patient for a single procedure.		= Manufacturer
	= Do not reprocess / re-sterilise		= Temperature range
	= Use by date		= Consult instructions for use
	= Lot or batch number.		= Keep away from sunlight
	= Date of manufacture. = Device catalogue number / product code / stock		= Keep dry
	= Caution		= Contains latex
	= Device catalogue number / product code / stock		= Do not use if packaging is wet or damaged.
			= Contains or Presence of Phthalates.

  
 1639 =CE mark with notified body number

• *This is an explanation of the symbols that may be found on Meditech Systems Ltd's products.*

• **ADVERSE INCIDENT REPORTING**

• Adverse incidents may be reported to the Competent Authority in the country / region where the incident occurred.

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